

LEVL SAFETY

TEST WITH CONFIDENCE

PROVIDER should use proper PPE - mask and disposable gloves.

1

PROVIDER inserts disposable mouthpiece into the LEVL breath pod and holds the breath pod for the patient to avoid cross contamination



2

PATIENT holds their breath for 5 seconds then pulls down their mask to exhale into the breath pod for 5-10 seconds. Patient returns mask to the proper position.

3

PROVIDER removes the disposable mouthpiece and places breath pod into the LEVL device for analysis. Results are delivered to the LEVL app in 15 seconds. Clean the breath pod and device with CDC approved disinfectant wipes.

DO NOT TEST PATIENTS WHO ARE SYMPTOMATIC OR WHO TEST POSITIVE FOR COVID -19



For more information, visit levlcare.com/safety.